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## 2 in 1 dans 1

**EN** **SIDE-FOLDING ROLLATOR & TRANSPORT CHAIR**

For proper assembly and to ensure your safety, follow these instructions.

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**PLEASE DO NOT RETURN THIS PRODUCT TO THE STORE.**

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If you're experiencing problems or have a question, contact our customer service team at CAN: 866-752-1363 USA: 866-694-5085.

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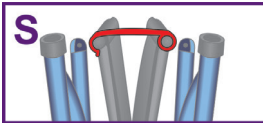
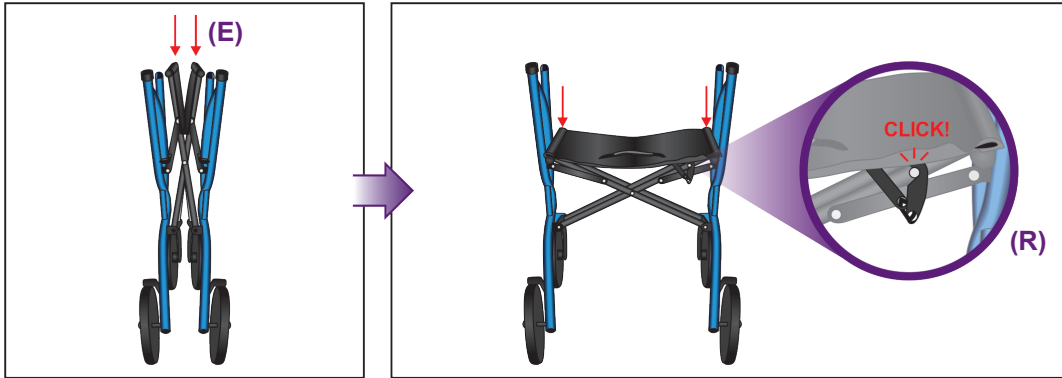
EN

Place the frame standing upright on level ground. Push down on the Seat Supports (E) to expand and open the frame. You will hear the Safety Lock (R) click into place when the frame is fully unfolded.

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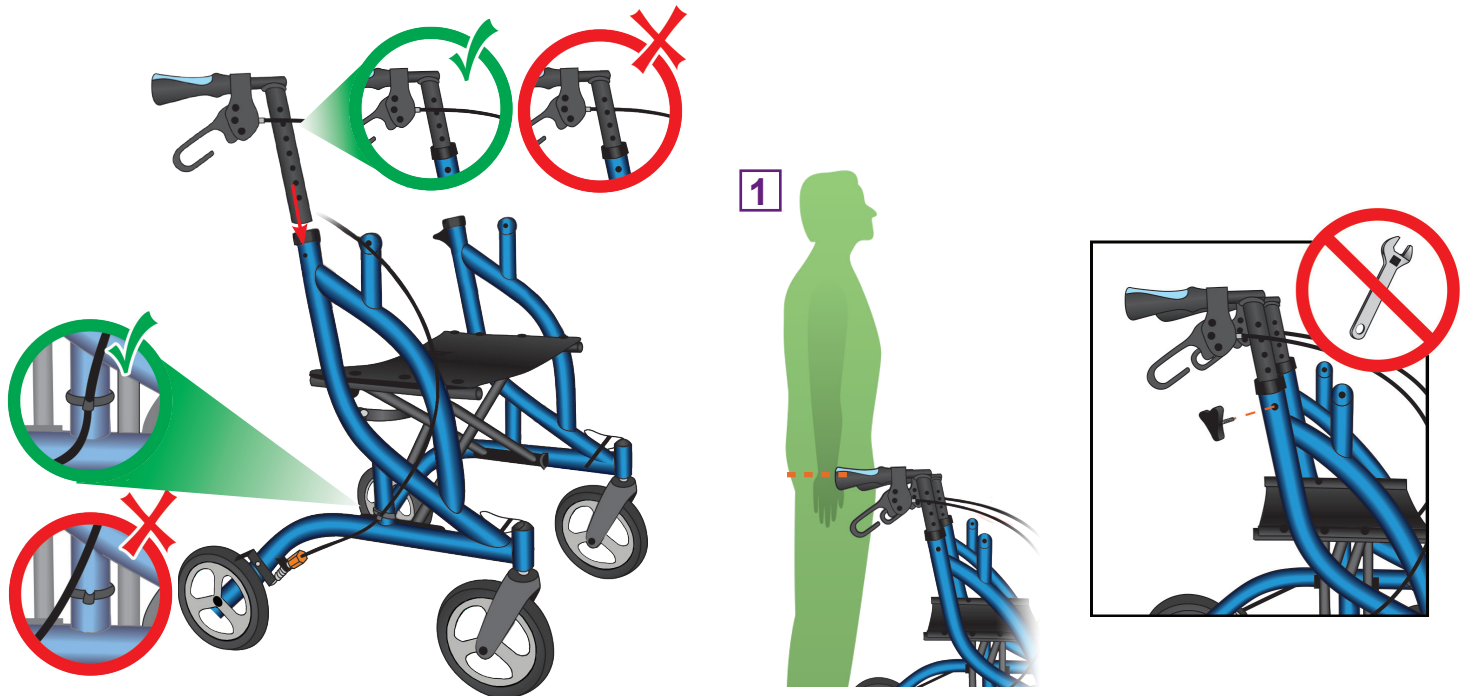
EN

Ensure the brake cables are on the outside of the frame, and that they pass through the cable guide. For height adjustment, stand upright and set the handgrips even with your wrists (1). Hand tighten the triangular hand screws.

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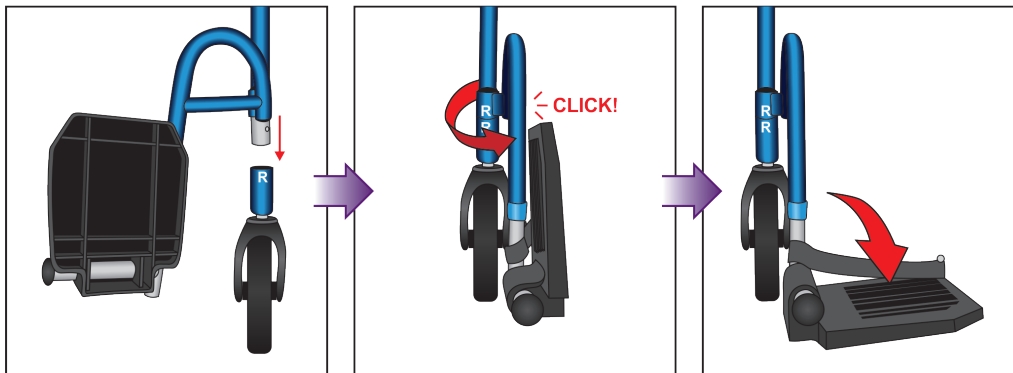
## INSTALL THE FOOTRESTS INSTALLEZ LES REPOSE-PIEDS INSTALACIÓN DEL REPOSAPIÉS

EN

Match the letters on the frame (*L and R*) to the same letters on the footrests. Press the push-buttons on the footrest tubing and insert into the Fusion's frame. The footrests can be locked in two different positions (front, for transport chair use, and side, for rollator use). To rotate the footrests, press the push-button and rotate.

When not in use, the footrests can be conveniently stored by pushing the push-button and swinging them to the side of the frame where another push-button will hold them securely in place. This allows you to lift and store your Fusion safely. The footrests can also be removed when not in use.

TIP: the black cap on the footrest tubing can be removed and used to cover the open tubing where the footrests slot into.

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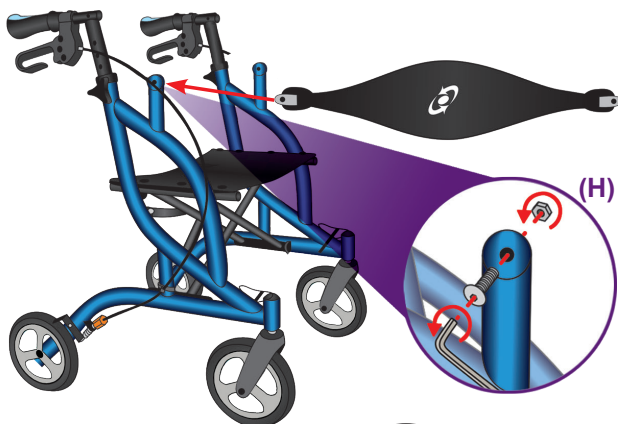
EN

Insert the backrest so that the Fusion logo faces the front wheels. Slide one end of the backrest into the fan bracket. Insert the backrest screw (H) from the inside of the unit out, and repeat. Install the basket by sliding it onto the mounts starting from the back, and then pulling to the front.

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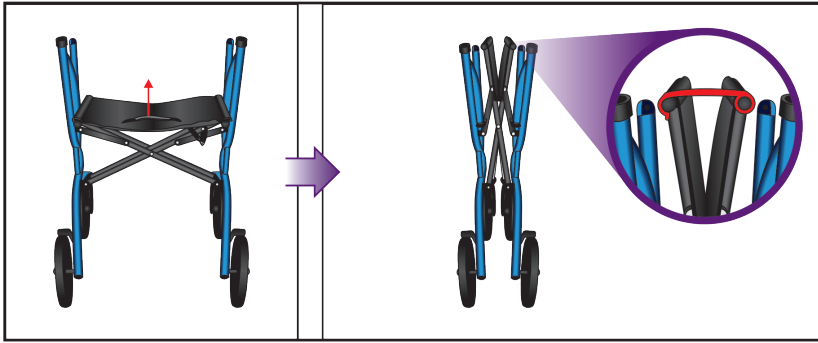
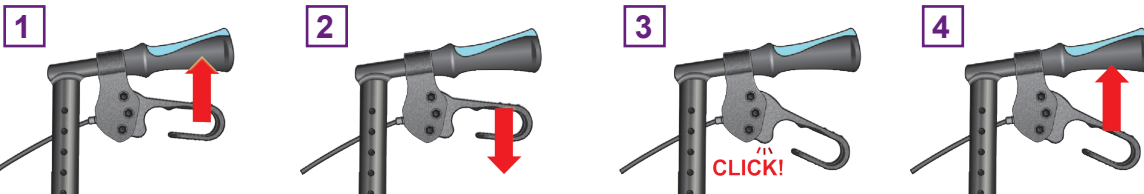
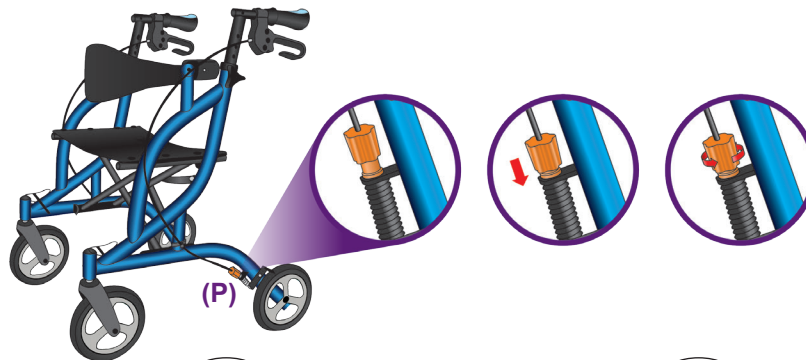
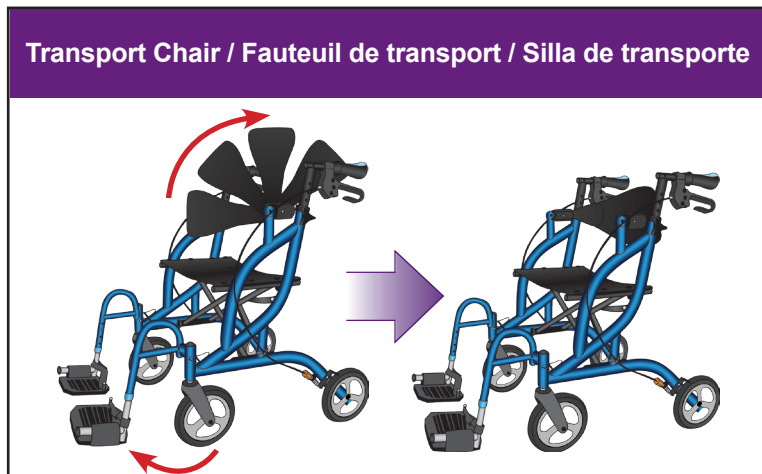
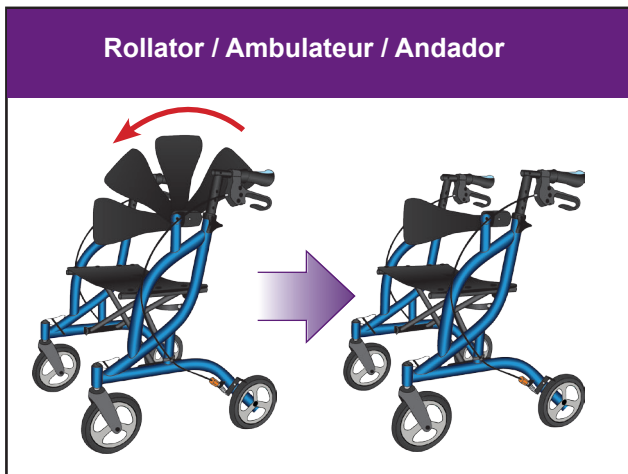
EN

### ROLLATOR MODE

Flip the backrest away from the handlebars so that it is positioned towards the front wheels. Before entering or exiting the seat always ensure the brakes are in the locked position by pushing down on both brake handles until you hear them click.

### TRANSPORT CHAIR MODE

Flip the backrest toward the handlebars so that it is positioned to the back of the Fusion. Then, for each footrest, press on the push-buttons above the front wheel and rotate the footplate toward the front of the Fusion until it locks into place. before entering or exiting the seat, always ensure that the footrest plates are folded up and that the brakes are locked. Once the user is seated, deploy the footrest plates as illustrated. Their height can be adjusted using the push-buttons on their tubing.

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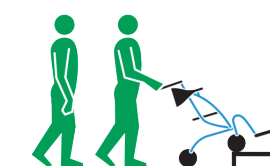
EN

### CAUTION WHEN SEATED

Leaning out of the Fusion without proper assistance could cause tipping. To reduce the risk of an accident, NEVER reach or lean when you are seated in your Fusion. Instead, have someone present to help you.

### CAUTION WHEN PUSHING

When pushing someone in the Fusion, do not roll across small uneven surfaces. Doing so can cause the unit to tip forward. To safely cross curbs, uneven surfaces or use on large inclines, the person sitting on the Fusion must get out of the seat.

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## WALKING TIPS CONSEILS DE MARCHÉ CONSEJOS DE MARCHÉ

EN

1. Imagine a line drawn between the rear wheels. Part of one foot should be ahead that line as you walk.
2. Perform simple stretches before walking. This will help you walk further and avoid injury.
3. Bring a bottle of water to stay hydrated.
4. Walk on soft ground that is more joint-friendly, such as grass or dirt paths.
5. Use the curb climber to help safely lift your rollator over obstacles, such as sidewalks.

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## READ THESE PRECAUTIONS PRIOR TO USING

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EN

1. Consult your doctor, therapist or other qualified professional to determine if this mobility aid will suit your particular needs.
2. Before each use, make sure that all parts are secure and that the frame is fully opened.
3. Risk of fall and serious injury may result if brakes are not locked while the user attempts to sit down.
4. Do not navigate independently while seated. Do not attempt to sit on the seat while on an incline. Do not lean when you are seated. Doing so may cause the walker to tip and a serious risk of injury may result.
5. Falls are possible if you use the seat without installing the backrest.
6. Do not use your rollator while folded. The stability of any rollator is compromised when its frame is folded, increasing the likelihood of a fall.
7. **Do not exceed the following weight limit or serious injury could result:**  
**Rollator Mode: 300 lb**  
**Transport Chair Mode: 300 lb**

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## LIMITED LIFETIME WARRANTY GARANTIE LIMITÉE À VIE GARANTÍA A VIDA LIMITADA

EN

Drive Medical covers this Airgo® rollator by a Lifetime Warranty on the Frame and a Five (5) Year Warranty on Brake Handles against defects in materials and workmanship. All other parts and components are warranted for a period of 90 days. This warranty is valid for the original purchaser only. Any alterations, abuse, misuse or accidental damage voids this warranty. Drive Medical reserves the right to replace or repair any part that has become defective.

For warranty service and general inquiries in Canada and the U.S., call our customer service team at **CAN: 866-752-1363 USA: 866-694-5085**. Provide the following information when you call: date of purchase, receipt and serial number (on crossbar underneath the seat). Our Customer Service Team will provide you with a return authorization number as well as all necessary shipping instructions. Note that with your return you will be required to include a copy of the original bill of sale.

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## AIRGO® RECOMMENDS AIRGO® VOUS RECOMMANDE AIRGO® LE RECOMIENDA

### Cup Holder

- A refreshment holder for your Airgo, to stay hydrated wherever you go!
- Swivels from left to right for maximum convenience.

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### Cane Holder

- Allows you to conveniently store your cane on your rollator.
- Easy to use, simply strap your cane along with you.

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Drive Medical LTD