EN Wash the tub where the Bath Safety Rail will be installed. All contact surfaces must be clean, dry and free of residue to ensure a secure attachment.

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EN This AquaSense® product is guaranteed to be free from defects in materials and workmanship for the life of the original user. This warranty does not extend to non-durable components which are subject to normal wear and replacement. Any alterations, abuse, misuse, or accidental damage voids this warranty. Rental of this product voids warranty. For replacement or repair under warranty, please call:
CANADA & INTL: 866-752-1363
USA: 866-694-5085
 For more information on this, or any other AquaSense® product, visit: www.myAquaSense.com

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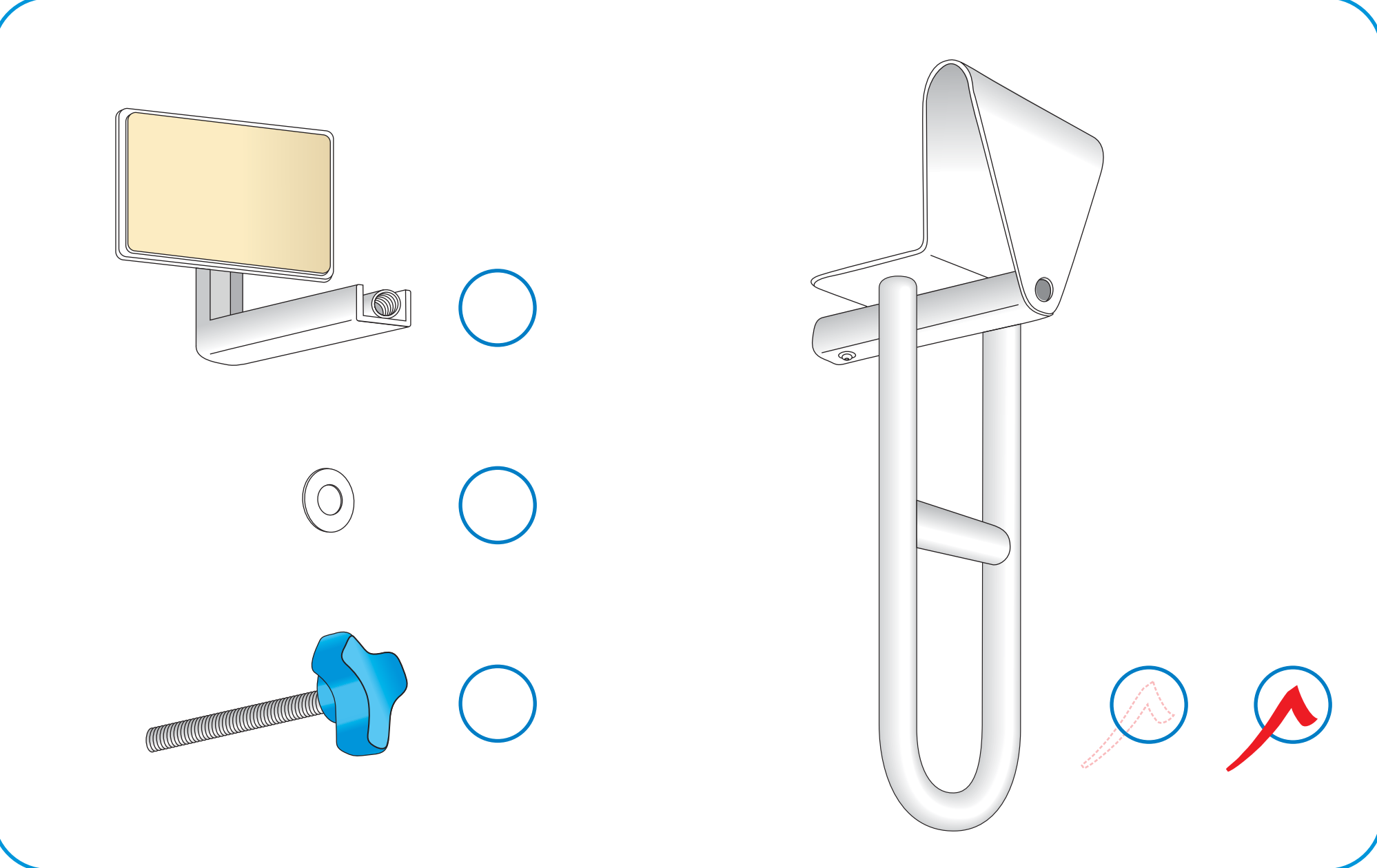
Assembled size: 27.9 cm x 21.6 cm when folded

Colours: _B/W X_CMYK _Spots:

Special Instructions: FRENCH FOLD (see diagram)

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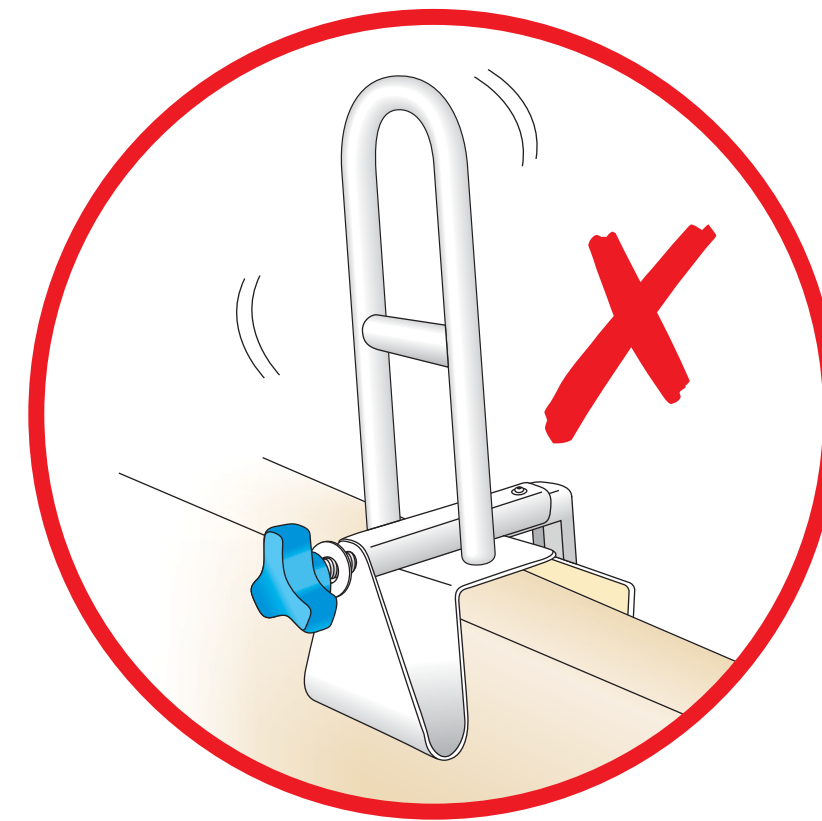
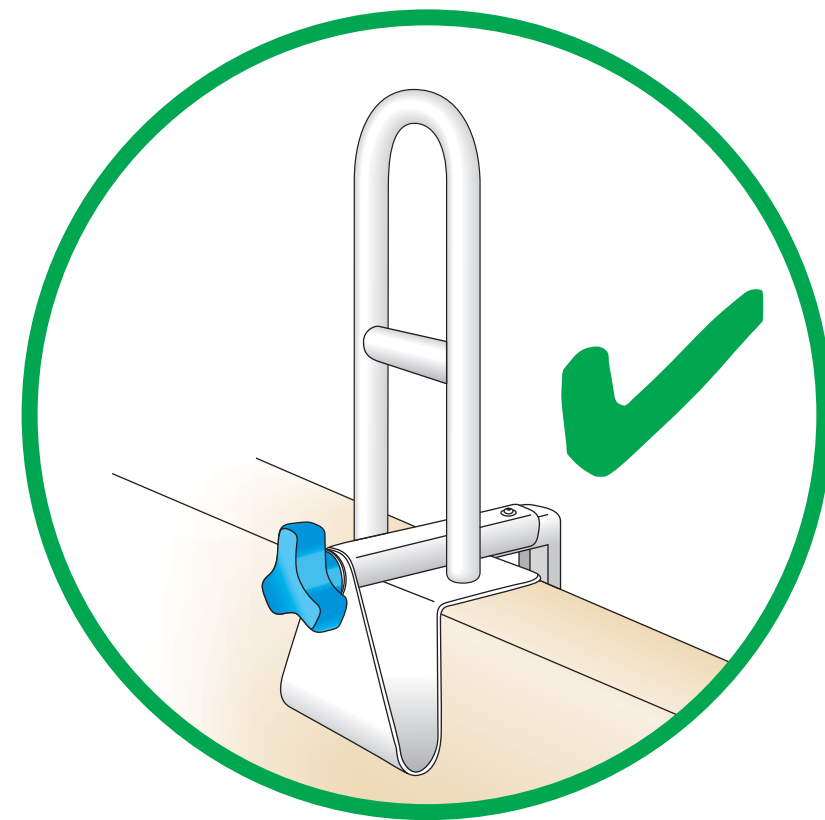
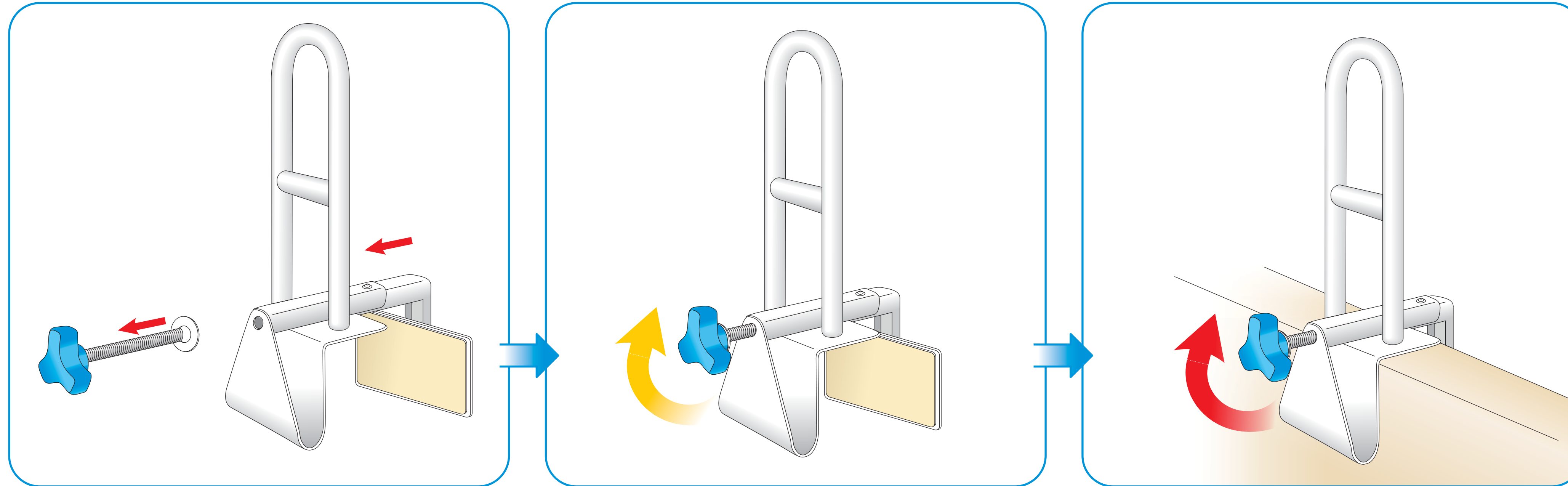
EN

- Tighten clamp carefully. Do not over-tighten on fiber glass and/or porcelain tubs as it may crack or damage them.
- Verify that the Bath Safety Rail is properly secured by pulling it from side to side and from front to back. If it moves, tighten again in the desired position. *Your safety rail may require tightening from time to time. Check periodically and tighten again when needed.*
- Use a mild household cleaner to maintain this product. Avoid the use of harsh solvents, abrasive powders and scouring pads.

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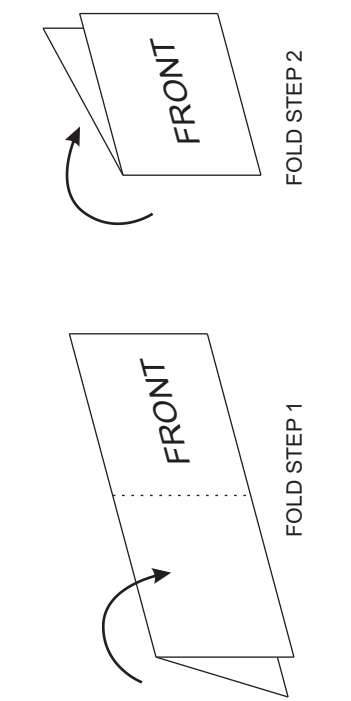
Project: 785-350 Instructions SIDE B
 Item No.: 785-350 Part No.: 921-785350
 Flat Size: 55.9 cm x 43.2 cm
 Assembled size: 27.9 cm x 21.6 cm when folded
 Colours: _BW X CMYK _Spots:
 Special Instructions: FRENCH FOLD (see diagram)
 Generated by: